



Tip sheet

Preparing the implant

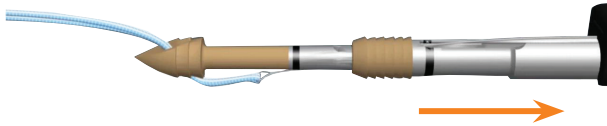

- When snaring sutures, pull straight back, then out
- Pull snare and sutures in direction of handle to avoid excess friction
- Ensure all sutures are in snare prior to deployment
- 1-2 inches is ideal for suture loading through the snare


Deploying the implant

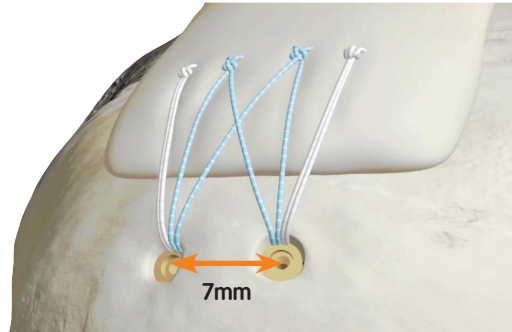

- Assure the eyelet of the anchor tip orients the sutures toward the tissue (ArthroCare logo should face the stitch)
- If laser mark is visible, re-align inserter handle and mallet further
- If placing more than one implant, space anchors at least 7mm apart

Deploying the implant, additional tips

- If issue occurs while driving implant tip into bone
 - Remove implant and create bone hole
 - If fractured, attempt to remove the tip
 - If unsuccessful, drill anchor tip with 3.3mm Spade Drill (ensure all debris is removed from joint space)
 - Replace with new MULTIFIX P anchor
- If fixation is compromised after sleeve engagement
 - Corkscrew implant may be a better option (i.e. 6.5mm SPEEDSCREW[®] or SPARTAN[®])
- When properly deployed, the first laser mark and PEEK will be visible through the driver window once separated

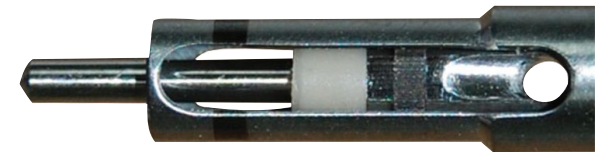

PEEK retained in guide/ inserter. This is ok.

Internal use only, do not distribute.